
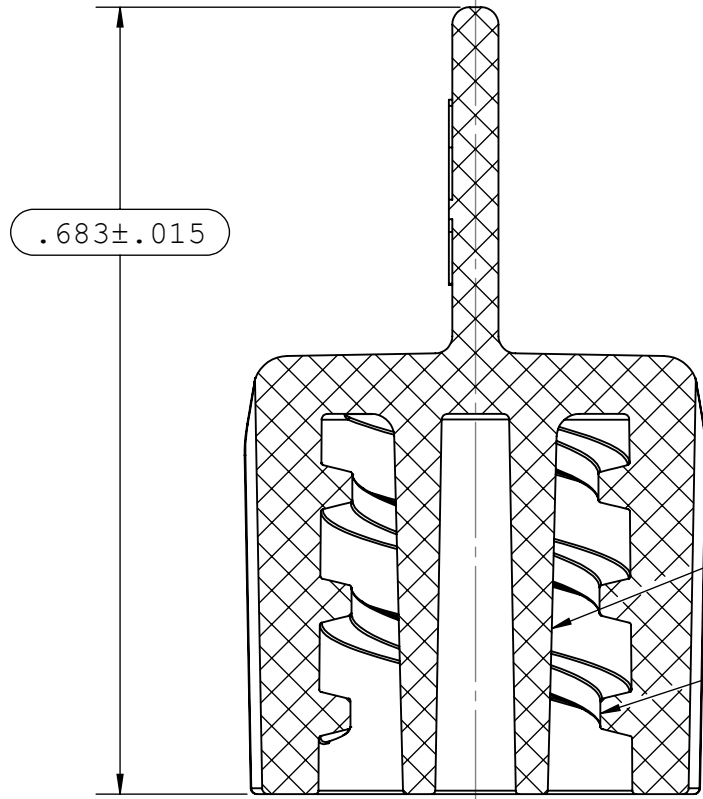


NOTES: FOR ADDITIONAL DETAILS, PLEASE CONTACT VP NEW PRODUCT DEVELOPMENT

1. DIMENSIONS MARKED  ARE CONTROL DIMENSIONS
2.  MUST USE CG102 INSPECTION GAGE FOR TAPER
3.  MUST USE S67 INSPECTION GAGE FOR THREADS



REVISIONS			
REV.	DESCRIPTION	DATE	APPROVED
A	INITIAL RELEASE	02-29-16	FJL
B	ECO 336	12-05-16	FJL

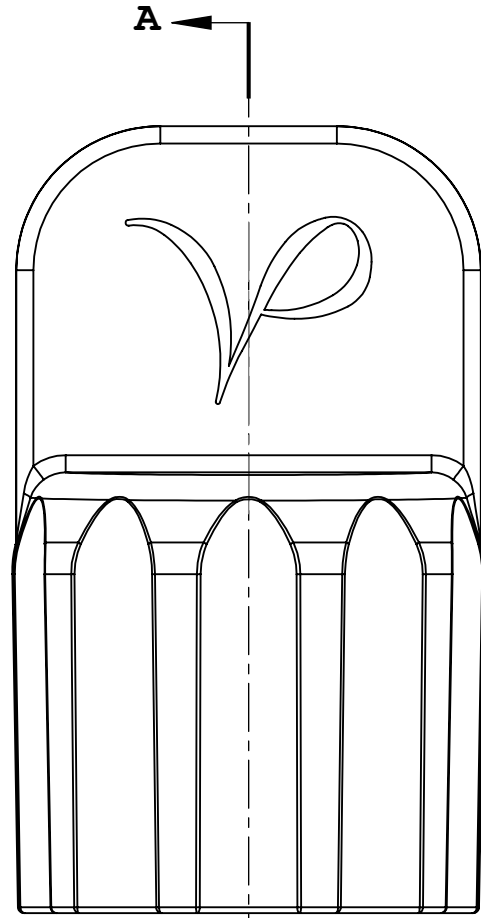

ISO 80369-6 TAPER

ISO 80369-6 THREAD

SECTION A-A

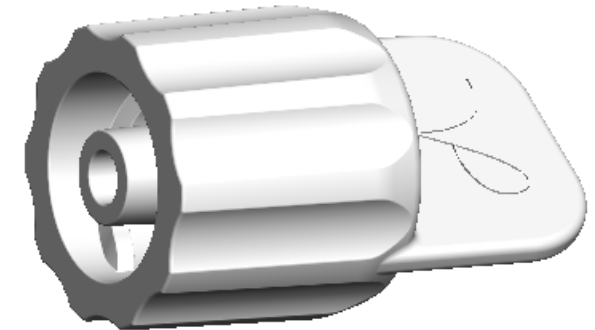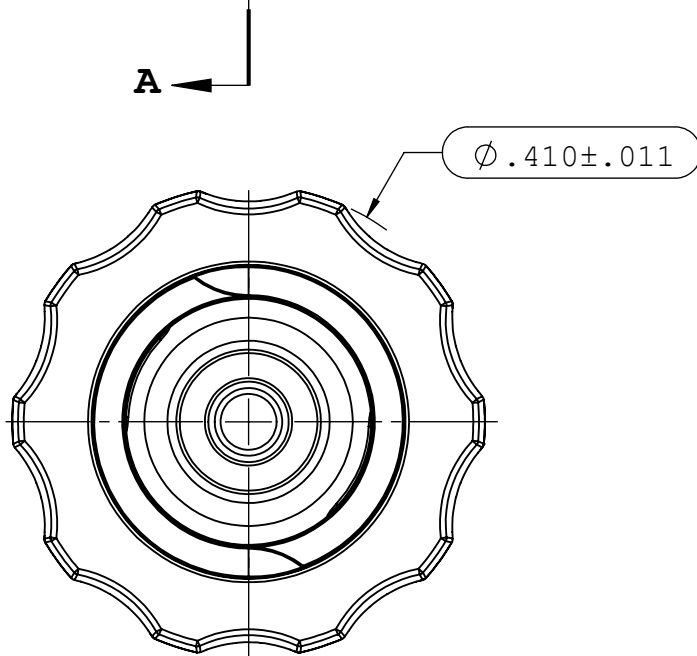

UNLESS OTHERWISE SPECIFIED
DIMENSIONS ARE IN INCHES
TOLERANCES ARE:

FRACTIONS	DECIMAL	ANGULAR
±	.XX ±.01	± 1°
	.XXX ±.005	
	.XXXX ±.001	

DO NOT SCALE DRAWING

VALUE PLASTICS, INC

PART NUMBER: NEMP

DESCRIPTION: Neuraxial Male Plug fits Female Connector

DRAWN BY: Pete L.	DATE: 12/5/2016	SIZE: B	SCALE: NTS	VER:	DWG. NO. 1039556	REV B
----------------------	--------------------	-------------------	---------------	------	---------------------	----------

MANAGER: Frank.Lombardi	© 2014 VALUE PLASTICS, INC	SHEET: 1 OF 1
----------------------------	----------------------------	------------------

FOR REFERENCE ONLY